

staple here

# Qseries

## how to use



**gomuQ Co.,Ltd.**

Zip 436-0079

TEL : +81-537-24-4027

50-18 kakegawa,kakegawa-shi, Shizuoka,Japan

FAX : +81-537-24-4028

Shizuoka,Japan

Mail : info@gomuq.com

<http://www.gomuq.com>

staple here

# Other tools

~Toothbrush~



1 Insert both spoon and thumb (up to base of thumb) into larger ring.



2 finished

~Spoon~



1 Insert both spoon and thumb (up to base of thumb) into larger ring.



2 finished

Above guidance is an example. Please try different positions and style that most fit to your requirement.

## Qring Point of choice

Age is reference only. Please select one that fits to your actual requirement.

	Qring 2 +soft (age: 2~4)	Qring 2 + (age: 2~4)	Qring 4 + (age: 4~8)	Qring 8 + (age: 8~)
Size of Finger hole				
material	soft/elastic	hard/elastic	hard/elastic	soft/elastic
color	light	dark	dark	light
Thumb size	small ←————→ large			



1 Insert pencil into smaller ring



2 Insert thumb into larger ring up to base of thumb



3 Finished



By placing pencil on first web, stable pencil gripping is possible

# Q ring

Q ring is a grip assistive tool for utensils like pencil and marker.

It is useful for people who tend to grab pencil or have difficulty.

It is also effective as training tool for kids who just began using pencil.


### Can be used

Commodities other than pencil, spoon for example, also can be gripped.



### 4 sizes 3 colors

Q ring offers 4 different sizes. According to the size of finger, select best fit size.

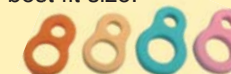

### Highly elastic

Highly elastic silicon rubber makes it easy to wear.



staple here

# Other applications

~Toothbrush~



1 Insert thumb (up to base of thumb) into larger ring and grip Q grip holder part.



2 Mark the holder point corresponding to palm end.



3 Take off Q grip and cut holder at marked point.



4 Insert toothbrush holder into smaller ring.



5 Insert thumb into larger ring and grip Q grip holder part.

~Spoon~



1 Insert thumb (up to base of thumb) into larger ring and grip Q grip holder part.



2 Mark the holder point corresponding to palm end.




3 Take off Q grip and cut holder at marked point.



4 Insert spoon holder into smaller ring.



5 Insert thumb into larger ring and grip Q grip holder part.

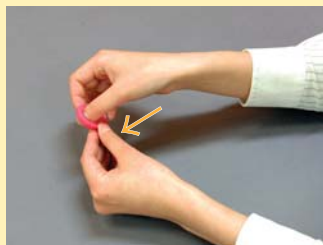

5 Insert thumb into larger ring (up to base of thumb).



6 Insert pencil into smaller ring.



7 Again, insert thumb into larger ring (up to base of thumb) Finish.



1 Insert thumb into larger ring up to base of thumb.



2 Grip Q ring holder by forefinger and little finger.



3 Mark the holder point corresponding to palm end.



4 Take off Q grip and cut holder at marked point.

# Q grip

Q grip is a grip assistive tool for utensils like pencil and marker.

In order to use Q ring effectively, you need to control five fingers independently.

Q grip is useful for people who have difficulty to achieve such control.

### Can be used

Not only gripping pencil, it can be used also for gripping other commodities.



### Two sizes

Q grip offers 2 different sizes. According to the size of finger, select best fit size.



### Highly elastic

It is so highly elastic that user will find it very easy to wear.



Above guidance is an example. Please try different positions and style that most fit to your requirement.



staple here

Q ruler15



### Easy to move

Knob is attached to top surface side for easy pickup and identification of press point.



### No slipping

Silicon rubber attached to bottom and top surfaces of ruler prevents slipping.



### Easy to read

Black and white ruler reading part realizes clear ruler reading.



### Easy to press

Unique bottom shape realizes easy pick-up and pressing.



# Q ruler15

Silicon rubber is attached to bottom and top surfaces of the scale; which eliminates slipping movement during use.

Users with disabled fingers or naturally all thumbs sometimes find it difficult to draw line by conventional slippery plastic or bamboo rulers.

As silicon rubber is attached to bottom and top surfaces of the Q ruler15, it is hard to move.

Just press the scale softly and the ruler has sufficient hold on base and straight line can be easily drawn.

Knob is attached to top surface side for easy pickup and identification of press point.



3

When you move ruler, pickup holder part.



2

When drawing line, press holder softly.



1

To be used on flat surface like desk top.



Point

Ruler can be put in conventional pencil case. You can prevent missing the ruler after use.



Point

Press front edge of the ruler and other end will be lifted up. You can pickup the scale easily.



Point

When bottom of ruler is dirty or dusty, clean it by wet towel and dry it before use.



Surface



### Heat resistive

Heat resistance up to 200°C. Will not stick to chair after long use.



### Nonslip function

Silicon rubber Q Chair Mat with nonslip function. Higher nonslip function than conventional Mats.



### Just fit size

Just fits to wooden school chair.



### No aching

Bottom surface of the mat is of special shape and makes it easy to pick-up and press.



# Q chairmat

Silicon rubber chairmat with nonslip function to be used with school chair.

Higher nonslip function than conventional mats.

It helps kids to keep correct sitting posture.

Wooden chairs used at school are hard and slippery; which makes kids difficult to keep correct posture and tired to remain seated.

Q chairmat improves such unfavourable situation for kids and enhances their study efficiency.

Q chairmat not only prevents slipping but also prevents hip aching from long time sitting with incorrect sitting posture.

Back surface

#### Point

Wooden chairs used at school are hard and slippery.



Slippery hip leads to incorrect posture.



Incorrect posture increases pain and leads to more incorrect posture.



Uneasy feeling decreases study efficiency.

**Q chairmat cuts this malignant cycle.**

By keeping correct posture for study, kids can improve their attention capacity and achieve easy handling of studying tools. As a result, they achieve better study efficiency.

2



Select chair with height enabling to touch both of your planters on floor. Place mat on the seat.

#### Point

Body weight should be on both ischial bones. Sit with pelvis upward posture.

3

Your back shall be straight up during sitting.

1



Place chairmat on seat.

#### Point




Back surface of the Q chairmat adopts honeycomb structure; which disperses body weight burden when sitting and reduces hip pain.



staple here

Free size 400×400mm

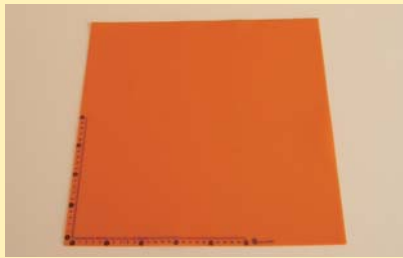

A4 size 324×232mm



# Q desksheet

## Wide application




Without pressing paper, you can use pencil or eraser.

## Nonslip function



Silicon rubber desksheet with nonslip function. Higher nonslip function than conventional sheets.

## Two sizes

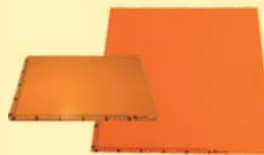

Two sizes A4 size and free size selection.

## Scale function




Scale is printed on bottom and left sides of the Q Desk Sheet.

Silicon rubber desksheet to be placed on desktop when studying. Higher nonslip function than conventional sheets. People with upper extremity disorder or hand find it difficult to fix paper by one hand.


They find it difficult to write, use eraser or scale.

By using Q desksheet, they can fix paper firmly by one hand.

Silicone sheet has proper hardness to be used as compass base or setting board.

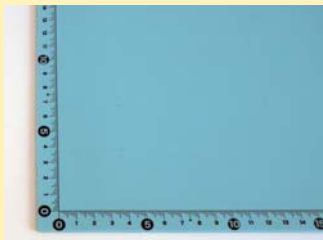

**Point** Measure is printed with special method and will not fade out under normal use.

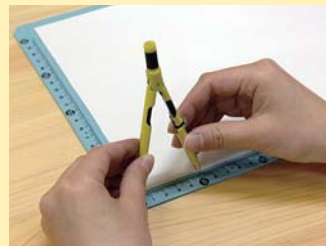


**Point** You can measure length using measure printed on the sheet edges. It is convenient to measure with compass.



**1** Place desksheet on flat surface like desktop.



**Point** It will be convenient to use with Q ruler15.

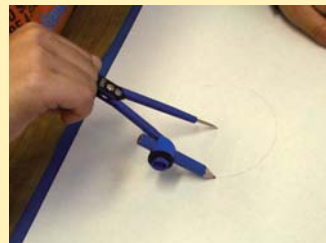

**Point** Q sheet has enough thickness to stick needle of a compass.



**Point** Without pressing paper, you can write or erase.



Qdesk sheet

staple here

M size for small press switch



L size for large press switch



Q switch

### Simple



Sucker is very easy to install.

### Two sizes



Two size selection according to the size of press switch to which Q switch is connected.

### Ladybird



Kids will find it fun to play with LADY-BIRD.

### Light touch input



Used with electronic press switch, input by different angle pressure is possible.

# Q switch

Q switch is used with press type switch.

Some handicapped person has difficulty to use press type input switch and needs arm fixing devices to realize switching by horizontal finger movement; which is complicated to wear and costly. It is easy to install by using sucker.

Just attach the Q switch sucker on top of the press switch. Then, switching by horizontal finger movement will be possible.



4

Put sucker on top of the center of press switch



3

Tear off isolation sheet of seal and put switch on silicon sheet



1

Place silicon sheet (Standard accessory) on flat surface like desktop



Point

Point: If sucking force become weak, wash sucker by water, exclude dust and reuse.

Point

To increase fixing strength, start using after 24 hours of this procedure



2

Stick switch fixing seal to bottom of input switch

